medela 

# symphony<sup>®</sup>

breastpump

bomba de lactancia

tire-lait

- ✓ Instructions for use
- ✓ Instrucciones de uso
- ✓ Mode d'emploi

## Table of Contents - English

Important Safeguards . . . . .	1
Uses . . . . .	2
Cleaning & Sterilization . . . . .	2
<i>Hygiene</i> . . . . .	2
<i>Preparation</i> . . . . .	2
<i>Hospital Sterilization</i> . . . . .	3
<i>Home Sanitizing</i> . . . . .	3
<i>After Each Use</i> . . . . .	3
Tubing Care . . . . .	4
Battery Care . . . . .	4
Symphony Parts List . . . . .	5
Symphony Assembly & Operation . . . . .	6-10
Symphony 2-Phase Pumping Protocol . . . . .	11-15
Storing Breastmilk . . . . .	16
Thawing Breastmilk . . . . .	17
Troubleshooting . . . . .	17
Symphony Technical Description . . . . .	18

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# IMPORTANT SAFEGUARDS

*When using electrical devices, basic safety precautions should always be followed, including the following:*

## READ ALL INSTRUCTIONS BEFORE USE

**DANGER:** To reduce the risk of electrocution:

1. Always unplug electrical devices immediately after using. Separation from the mains is only assured through the disconnection of the plug and socket connection.
2. Do not use while bathing or showering.
3. Do not place in or drop motor unit into water or other liquid.
4. Do not reach for any electrical device if it has fallen into water. Unplug immediately.
5. Do not store or use this product where it can fall or be pulled into a tub or sink.

**WARNING:** To reduce the risk of burns, electrocution, fire or injury to persons:

1. The pump should never be left unattended when plugged into an electric outlet.
2. Use the pump only for its intended use as described in this manual.
3. Do not use accessory attachments not recommended by the manufacturer, as this may cause hazards.
4. Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water.
5. Keep the cord and all components away from heated surfaces.
6. Do not use outdoors.
7. Only use the electric cord that comes with the Lactina or Symphony Breastpump.
8. The kits are personal care items and are not to be shared between mothers without proper sterilization to prevent the risk of cross contamination.
9. The pump should be cleaned as needed with soapy water or Cavicide® solution. Do not spray or pour liquid directly onto the pump.
10. Supervision is necessary when this product is used in the vicinity of children.
11. Make sure the voltage of the device is compatible with the power source.
12. Pump only with the motor in an upright position.
13. Do not use while sleeping or overly drowsy.
14. There are no user-serviceable parts inside the Lactina® or Symphony® Breastpump. DO NOT ATTEMPT TO REPAIR THE LACTINA OR SYMPHONY BREASTPUMP YOURSELF.

**SAVE THESE INSTRUCTIONS**

## Uses

- To initiate and maintain a milk supply as well as collect breastmilk at work or during other absences from the baby.
- To initiate and maintain a milk supply if direct breast-feeding is not possible.
- To relieve engorgement. Single pumping with simultaneous massage is helpful to relieve engorgement.

## Cleaning and Sterilization

*Unless instructed otherwise by your health care provider.*

### IMPORTANT

*This product's kit is a single-user item unless properly sterilized between users. Use by more than one user without adequate sterilization may pose a health risk and voids the warranty.*

### Hygiene

Wash hands before touching breasts or containers, and avoid touching the inside of containers or lids.

### Preparation

Follow these cleaning instructions unless you are told otherwise by your healthcare professional. Prior to cleaning, take apart all parts of your breastpump accessory kit. Be especially sure to remove the yellow valve from the breastshield and separate the white membrane from the valve prior to cleaning.

## Prior To First Use

The Symphony sanitized or sterilized kit versions do not require washing before first use.

---

### ***To Sterilize in the Hospital:***

All disassembled parts can be autoclaved to a maximum of 272° F for 3 minutes at 29 PSIG or 250° F for 15-20 minutes at 15 PSIG. If autoclaved membrane caps may require more force to attach to pump.

To avoid damage during and immediately after sterilization:

- Sterilize parts without applying external pressure to them.
- Allow parts to cool before applying external pressure.

If gas sterilization is used, residual testing is recommended to determine the proper length of aeration.

### ***To Sanitize at Home:***

Disassemble, refer to diagram on page 6. Sanitize by boiling for 10 minutes.

### ***After each use:***

- Disassemble and wash all parts that come in contact with the breast and milk in warm soapy water.
- Rinse in clear water.
- Air-dry on clean towel and cover parts when not in use.

All parts may also be washed in top rack of dishwasher.

### ***For Quicker Cleaning:***

Medela offers Quick Clean™ Micro-Steam™ Bags for daily kit disinfection.

### **Tubing Care:**

Inspect tubing after each pumping session for condensation and or milk.

---

### ***If condensation appears in tubing:***

To eliminate condensation after you have completed pumping, continue running the pump with the tubing(s) attached for another 1-2 minutes or until dry, while you attend to storing your pumped breastmilk.

### ***If milk appears in tubing:***

1. Turn off the pump and unplug from power source.
2. Remove and disassemble the tubing, protective membrane and membrane cap. Wash in soapy water then rinse in cold clear water.
3. Shake water droplets out and hang to air-dry. For faster drying, attach tubings to pump and run pump 1-2 minutes or until dry, or pour a small amount of isopropyl alcohol through the tubings to dry.
4. Make sure that valve membranes are undamaged and clean. Replace if necessary.
5. Use a damp cloth to wipe the pump and the diaphragm. Do not immerse the pump in water.
6. When components are completely dry, reassemble breastpump and kit.

## **Important Battery Care**

### **Charge Symphony® pump batteries for 12 hours upon receipt.**


Pump may be used while charging. Optimal battery life is achieved when:

- Batteries are not fully drained on a regular basis
- Pump is allowed to charge when not in use

If unable to charge regularly, charge for 12 hours every 2 months at minimum

Pumping time with a completely charged battery is approximately 60 minutes.

# Symphony® Breastpump Parts

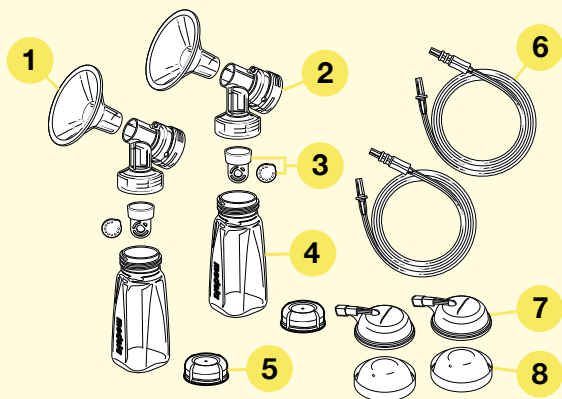


Item	Order Number
1. PersonalFit™ Breastshield, Standard (24mm).....	87073
2. PersonalFit Breastshield Connector .....	87071
3. Valve & Membrane.....	8000001
4. Collection Container.....	8117009
5. Solid Container Lid .....	8107183
6. Tubing .....	8007213
7. Membrane Cap.....	8100769
8. Protective Membrane.....	8200094
9. Breastfeeding Information Guide.....	1907200
10. Symphony Instruction Manual.....	1907724
11. Symphony Container Stand .....	8100552
12. Symphony Vehicle Lighter Adapter .....	67173

**Call for pricing**

# Symphony® Assembly & Operation Kit Assembly

1. Follow cleaning instructions on pages 2-3.
2. Insert the small end of the Breastshields into the small end of the connectors. (Prior to cleaning, separate these two pieces.)
3. Snap white membranes onto yellow valves until membrane lies completely flat. (Prior to cleaning, separate these two pieces.)
4. Push assembled valves and membranes onto the breastshield connector.
5. Screw containers onto the breastshield connector.
6. Insert hard yellow ends of tubing into openings in the breastshield connector.
7. Assemble protective membranes onto membrane caps. Push triangular ends of tubing into the small openings in the membrane caps.

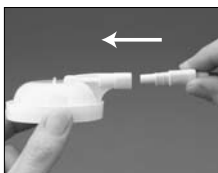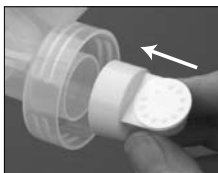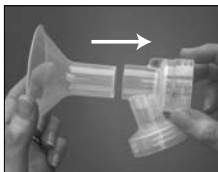
8. To unlock case, press the oval release button on top of the Symphony® Breastpump. Keeping button depressed, lift up on the handle to open case and access pumping mechanisms.

9. Push membrane/cap into place on pumping mechanism. A “click” will be heard once properly seated. To ensure proper fit, be sure to center the membrane caps’ tube ports directly over the crevice on the pump housing.

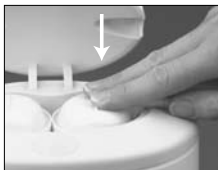
10. Power the Symphony Breastpump (or charge its battery if included) by connecting the power cord to a wall outlet and the socket in the back of the pump.

**Important Note:** Kits sold separately. For a complete list of kits that are compatible with the Symphony pump, please refer to the Medela Product Catalog or contact Customer Service at 1-888-633-3528.

**Important Note:** If you experience discomfort, contact your lactation professional as you may need a larger breastshield or other advice. To locate Medela products or a breastfeeding specialist in your area call 1-800-TELL YOU.

# Symphony® Controls

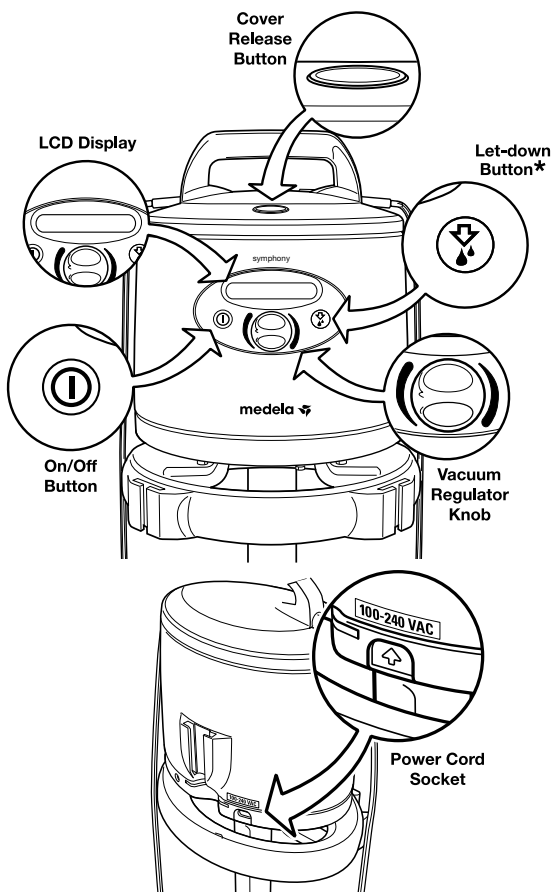\* (Press if let-down occurs before expression phase starts, when using the Standard 2.0 program. See page 10, step 4 for specific details.)

# Symphony® Program Cards

The Symphony has the unique ability to be updated with new clinical research as it becomes available. There are two versions of the Symphony Card: The Symphony Standard 2.0 Card and the Premie+™ Card. The following information includes instructions for either card.

## ***Symphony Standard 2.0 Card***

The Symphony Standard 2.0 Card is the second generation card. It is programmed to operate the 2-Phase Expression® technology. There are other important features available with the Symphony Standard 2.0 Card:

### **1. *Card Name Displayed on LCD Display.***

The Symphony Standard 2.0 Card displays the name of the card (Symphony 2.0) on the LCD display when the pump is powered off and connected to A/C power supply. When batteries are used to power the Symphony, the LCD Display will remain blank until powered on.

### **2. *Minimum Start Vacuum Setting.***

The Symphony Standard 2.0 Card will start the pump in the Stimulation Phase. The initial vacuum setting when powering on the pump is visually indicated with 3 drops on the LCD screen. The vacuum setting can be increased or decreased by using the Vacuum Regulator Knob.

### 3. **Ramp-Up Transition from Stimulation to Expression Phase.**

When switching from Stimulation to Expression Phase, the Symphony Standard 2.0 Card increases the vacuum setting by adding 3 bars to the last Stimulation setting in a stair-stepped fashion. The 3 additional bars and increased vacuum setting occur gradually over a 4-second period. The ramp-up transition eliminates potential discomfort of an immediate vacuum setting increase.

### 4. **Use Let-Down Button to Switch Back to Stimulation Phase.**

The Symphony pump can switch from Stimulation to Expression Phase in one of two ways: 1) automatically after 2 minutes, or 2) manually through pressing the let-down button. With the Symphony Standard 2.0 Card, the let-down button can also be used to switch back to Stimulation Phase. For more information on this feature, please review the Symphony 2-Phase Pumping Protocol information in this booklet.

**Note: Stimulation Phase is always limited to two minutes. If milk ejection occurs before the 2 minute Stimulation Phase is over, press the let-down button and continue pumping in Expression Phase**

### 5. **Program Remembers Last Stimulation Vacuum Setting.**

When switching back to Stimulation Phase, the Symphony Standard 2.0 Card remembers the last Stimulation Phase vacuum setting prior to switching to Expression Phase and returns to this setting in Stimulation Phase.

# Symphony® 2-Phase Pumping Protocol

## **Maximum Comfort Vacuum™**

Research has shown that you will pump more efficiently—meaning get more milk in less time—when pumping at Maximum Comfort Vacuum in the Expression Phase.

### **1. Begin in Stimulation Phase**

First, start the pump by pressing the on/off button. The pump will immediately begin pumping in Stimulation Phase of the 2-Phase.

### **2. Switch to Expression Phase**

Once your milk starts to flow, or you sense milk ejection, press the let-down button to switch to the Expression Phase for the most effective pumping and removal of milk. If you do not press the let-down button, the pump will automatically switch to the Expression Phase after two minutes.

### **3. Setting your Maximum Comfort Vacuum**

Maximum Comfort Vacuum is the highest vacuum you can use and still feel comfortable; it is different for every mom. To find your maximum comfort vacuum, increase the vacuum by turning the Vacuum Regulator Knob clockwise until pumping feels slightly uncomfortable—never painful. Then, decrease the vacuum one bar by turning the knob counterclockwise.

---

## ***Symphony Standard 2.0 Card— Addressing Special Needs***

The Symphony Standard 2.0 Card includes a feature that allows moms to switch back to the Stimulation Phase by pressing the let-down button. Some moms find this feature beneficial in addressing the following special needs. Please note that the Stimulation Phase is always limited to two minutes.

### ***1. No milk ejection occurs in the first two minutes of the Stimulation Phase***

If your milk ejection has not occurred in the first two minutes of the Stimulation Phase and the pump automatically switches to the Expression Phase, press the let-down button once to return to the Stimulation Phase. If you have not achieved a milk ejection after two consecutive Stimulation Phases, take a break, relax and try again in 15-30 minutes.

### ***2. Engorgement***

If, due to engorgement, you are experiencing difficulties breastfeeding or expressing milk, careful use of the Stimulation Phase may help to provide some relief. If your milk ejection does not occur in the first two minutes of the Stimulation Phase and the pump automatically switches to the Expression Phase, press the let-down button to return to the Stimulation Phase. Repeat this process until your milk ejection occurs. After milk ejection you may find it beneficial to remain in the Stimulation Phase until enough milk has been expressed so that the nipple and areola area are not as hard and engorged. After initially draining the breast to relieve the discomfort, follow up with breastfeeding or pumping and expressing in the Expression Phase to adequately drain the breast. Although Medela strongly encourages pumping in the Expression Phase after milk ejection at Maximum Comfort Vacuum™ for the most effective removal of milk, it may be necessary to address temporary discomfort issues by pumping in the Stimulation Phase. If your engorgement condition

---

persists or worsens, consult your physician or lactation consultant.

### 3. **Sore Nipples**

If, due to sore nipples, you are experiencing discomfort while pumping in the Expression Phase, careful use of the Stimulation Phase may help to provide some relief. You can return to the Stimulation Phase at any time by pressing the let-down button. Although Medela strongly encourages pumping in the Expression Phase after milk ejection at Maximum Comfort Vacuum™ for the most effective removal of milk, it may be beneficial to address temporary discomfort issues by pumping in the Stimulation Phase. If your sore nipple condition persists or worsens, consult your physician or lactation consultant.

**Important Note:** For the most effective pumping and removal of milk, Medela strongly encourages pumping in the Expression Phase after milk ejection at Maximum Comfort Vacuum™.

## **Symphony® Preemie+™ Card**

The Symphony Preemie+ Card provides two programs for breastpumping (the Preemie+ 1.0 program and the Standard 2.0 program). The combination of the two programs has been clinically shown to allow mothers of premature infants to provide significantly greater milk output in fewer minutes of pumping. The Preemie+ 1.0 program is designed to get pump dependent mothers started breastpumping. Its new unique burst/pause pattern is clinically shown to help mothers of premature infants initiate milk flow as it mimics baby's first sucking pattern after birth.

### **Initiation—Preemie+ 1.0**

**Important Note:** You should only use the Preemie+ 1.0 program until you achieve 3 consecutive pumping sessions, each totaling 20 mls. Once you have achieved this, you should switch to the Standard 2.0 program.

1. When plugged in the name reflects that both the Preemie+ 1.0 program and the Standard 2.0 program are on the pump. Press On/Off to start pumping.

2. Center breastshields over nipples.

3. Symphony displays “💧 for Preemie+™”. If you desire to start the Symphony Preemie+ 1.0 program, you should press the drop or “let-down” (👉) button. This is displayed for 5-10 seconds.

4. “Start Preemie+” is displayed.

5. After a 5 second delay, 1 drop will be displayed. You can now pump with the Symphony Preemie+ 1.0 program. The pumping session lasts for 15 minutes. You should use it for the entire 15 minute duration.

## 2-Phase Expression®— Standard 2.0

The Standard 2.0 program should be started once 3 consecutive pumping sessions, each totaling 20 ml, have been achieved. Again, the combination of the two programs has been clinically shown to allow mothers of premature infants to provide significantly greater milk output in fewer minutes pumping.

1. When plugged in, the name reflects that both the Premie+™ 1.0 program and the Standard 2.0 program are on the pump. Press On/Off to start pumping.

2. Center breastshields over nipples. Symphony displays "💧 for Premie+™". to start the Symphony Premie+ Program. To start the Standard 2.0 program, wait for display to read "Start Standard".

3. After a 5 second delay, drops will be displayed. You can now use the Standard 2.0 program to begin the Stimulation phase.

4. If your let-down occurs before the Expression phase begins, you may pump more efficiently by pressing the Ⓣ button to begin the Expression phase.

5. If you need to return to stimulation for another let-down or did not achieve let-down after 2 minutes, simply press the let-down button to return to stimulation for another 2-minute cycle.

# Storing Breastmilk

Check with your hospital for specific storage instructions.

When freezing, do not fill containers more than 3/4 full to allow space for expansion. Label containers with the date of pumping.

Freshly Expressed Breastmilk Storage Guidelines (For Healthy Term Babies)				
Room Temperature	Cooler with 3 Frozen Ice Packs	Refrigerator	Freezer	Thawed Breastmilk
4–6 hours at 66–78°F (19–26 °C)	24 hours at 59 °F (15 °C)	3–8 days at 39 °F or lower (4 °C)	6–12 months 0–4 °F (-18–20 °C)	use within 24 hrs

References: [www.BreastmilkGuidelines.com](http://www.BreastmilkGuidelines.com)

Hamosh M, Ellis L, Pollock D, Henderson T, and Hamosh P: *Pediatrics*, Vol. 97, No. 4, April 1996. pp 492-497. (4 hours at 77° F/25° C).

The Academy of Breastfeeding Medicine Protocol Committee. *ABM Protocol #8: Human milk storage information for home use for healthy full-term infants.* 2004.

Adeola, K.F., Otufowora, O.A. Effect of Storage Temperature on microbial quality of infant milk. *J Tropical Peds* 1998 Feb; 44(1): 54-55.

Hands, A. Safe Storage of expressed breast milk in the home. *MIDIRS Midwifery Digest* 2003; 13(3):378-85.

Jones, F. and Tully, M.R. *Best Practice for Expressing, Storing and Handling Human Milk in Hospitals, Homes and Child Care Settings, Second Edition.* The Human Milk Banking Association of North America, 2006.

Lawrence, R. and Lawrence, R. *Breastfeeding: A Guide for the Medical Profession, Sixth Edition.* St. Louis; Mosby, 2005; 1018-20.

Martinez-Costa, C., Silvestre, M.D., and Lopez, M.C. et al. Effects of refrigeration on the bactericidal activity of human milk: A preliminary study. *J Pediatr Gastroenterol Nutr* 2007; 45:275-77.

**Important Note:** If the temperature in the room, car or outdoors exceeds 77° F / 25° C, chill milk immediately to preserve freshness.

## Defrosting/Warming

- You can use the Medela® Waterless Milk Warmer™ (if available).
- You can thaw breastmilk in the refrigerator for up to 24 hours. **Do not refreeze.**

---

## **Caution:**

- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water.
- Never microwave breastmilk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. Microwaving can also change the composition of breastmilk.
- If adding expressed breastmilk to a container of already frozen breastmilk, make sure to add a lesser amount than the already frozen amount.

## **Troubleshooting**

### ***If There Is Low or No Suction***

- Make sure all connections of kit and breastpump are secure.
- If using a unit with a battery, make sure the battery is charged.
- Make sure membrane is lying flat against valve head. Refer to the assembly instructions.
- Make sure the hard yellow ends of the tubing fit securely into breastshield connectors and the other ends into the ports on the pump connector or membrane caps.
- In the US, call Medela at 800-435-8316 or 815-363-1166. In Canada, call Medela at 800-435-8316 or 905-608-7272. In other countries, contact your local distributor or the location where you purchased your pump. Visit us on the internet at [www.medela.com](http://www.medela.com).

# Symphony® Technical Description

## SAFETY INSTRUCTIONS

Medical Electrical Equipment, Breastpump Model Symphony® 600.0837 and 600.0838  
**CLASSIFIED WITH RESPECT TO ELECTRIC SHOCK, FIRE AND MECHANICAL HAZARDS ONLY IN ACCORDANCE WITH UL60601-1/CAN/CSA C22.2 No. 601.1 22PL.**

## PROTECTION CLASS II

(DoubleInsulation)

Type B

## VACUUM:

-50mmHg – -250mmHg  
-7kPa – -33kPa

## POWER SUPPLY:

### Conventional Outlet:

100-240 V  
50/60 Hz  
0.5 A

### DC Connection:

12 VDC  
25 W  
T2.5A

### Battery:

2 x 6 V, 1.2 Ah Type: Yuasa NP 1.2-6  
Battery(optional): Battery,  
DC powered  
Charging time=12 hours  
Full battery pumping  
time=approx. 60 minutes

## DIMENSIONS:

*With Stand    Without Stand*

Length	16.1"	8.5"
Width	16.1"	6.1"
Height	35.4"	9.6"

## WEIGHT:

With Battery: 6 lbs. 7 oz.  
Without Battery: 5 lbs. 1 oz.

## ENVIRONMENTAL CONDITIONS FOR OPERATION:

Temperature: +41°F to +104°F  
Relative Humidity: 20% to 80%  
Atmospheric Pressure: 70 kPa to 106 kPa

## ENVIRONMENTAL CONDITIONS TRANSPORT AND STORAGE:

Temperature: -4°F to +122°F  
Relative Humidity: 20% to 95%  
Atmospheric Pressure: 70 kPa to 106 kPa

## STANDARDS:

FDA (GMP)  
ISO 9001  
ISO 13485  
CE (93/42/EEC)

## ELECTROMAGNETIC OR OTHER INTERFERENCE:

Complies with IEC 60601-1-2. Unlikely to interfere with other electrical equipment.

## DISPOSAL:

This breastpump is made of technical plastics and metals that cannot be separated. It must be disposed of in accordance with local regulations.

## WARRANTY:

3-years for the unit from the date of delivery, excluding pump set and batteries. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage or use by unauthorized persons. Routine checks, servicing and battery exchange may only be carried out by Medela authorized personnel.

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Medela, Inc., P.O. Box 660,
Medela Canada, Inc.,