

MICROBAN® TRI-PANEL KNEE IMMOBILIZER

Sizes: Universal
Color: Blue/White

Model No:

- 38-140 12"
- 38-141 14"
- 38-142 16"
- 38-143 18"
- 38-144 20"
- 38-145 22"
- 38-146 24"
- 38-147 26"

TO DETERMINE SIZE:

First select the appropriate length for the particular patient. The immobilizer should not dig into the groin or reach the ankle bone. As a guide, lengths 18"-20" typically fit medium adults, and the 24" length typically fits tall adults. Measure around leg 4" above center of knee.




Size	Fits
Universal	13 - 26"

Popular universal knee immobilizer has adjustable side panels to fit a range of sizes. Treated with Microban® anti-microbial protection for a more hygienic product. Constructed of a medical grade foam with a soft loop liner for patient comfort. Lateral and medial stays increase stability. Immobilization created with contoured posterior stays. Easy loop lock closures. Universal size.

Care Instructions: Fasten closures and hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

Wear Instructions: The tri-panel style has adjustable side panels to fit a wide range of sizes and panels can be angled for a tapered fit.

1. Determine the appropriate width by removing the medial and lateral panels and placing in the desired position. Placing the panels at an angle will improve the fit for wider thighs and thinner calves. The medial and lateral stays should still be aligned with the sides of the leg. Ensure the hook on the side panels is fully attached to the base.
2. Place immobilizer behind knee using the contoured posterior stays as a guide for proper positioning. The inverted portion of the stay(s) should be placed in the popliteal area (directly behind the knee). Ensure the medial and lateral stays are aligned with the side of the leg.
3. Close the base of the immobilizer, then fold around and close the outer panels.
4. With the immobilizer in correct position, start securing straps immediately above and below the patella first.
5. Work from the bottom upwards, closing the additional straps. Use the removable piece of double sided hook for a more customized fit. Excess strap length may be trimmed to fit.
6. Adjust if necessary.

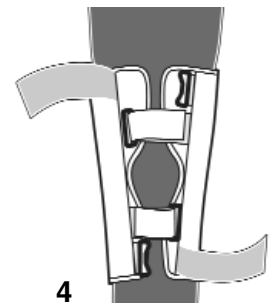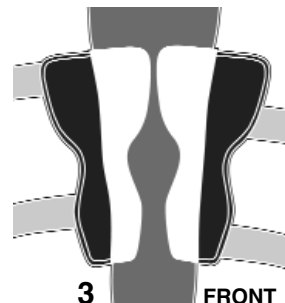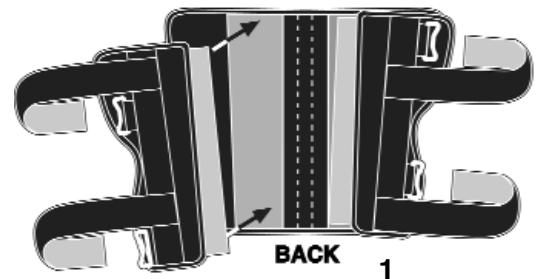

800-327-4110



Website: www.flaorthopedics.com

©2004 FLA ORTHOPEDICS, INC.