## Contents

<b>Instructions for the Ameda® HygieniKit Milk Collection System</b>	<b>2</b>	<b>Converting to an Ameda One-Hand Manual Breast Pump</b>	<b>7</b>
<i>Intended Use</i>	2	<i>Assembly</i>	7
<b>Set Up and Assembly</b>	<b>2</b>	<b>Using the One-Hand Manual Breast Pump</b>	<b>8</b>
<i>Parts Diagram</i>	2	<i>Pumping Made Simple</i>	8
<b>Cleaning</b>	<b>3</b>	<i>Removing Ameda One-Hand Breast Pump from Breast</i>	8
<i>Before First Use</i>	3	<i>Troubleshooting</i>	8
<i>After Every Use</i>	3	<b>Milk Storage and Handling</b>	<b>9</b>
<i>Sterilizing in the Hospital</i>	3	<i>Containers</i>	9
<b>Using the Ameda HygieniKit Milk Collection System</b>	<b>4</b>	<i>Guidelines for Storing Human Milk</i>	9
<i>Attaching to Any Ameda Electric Breast Pump</i>	4	<i>Thawing/Warming Milk</i>	9
<i>Single and Dual Pumping</i>	4	<b>Flexishield Areola Stimulator</b>	<b>9</b>
<i>Putting Ameda HygieniKit to the Breast</i>	4	<b>Replacement Parts</b>	<b>10</b>
<i>Checking Flange Fit</i>	4	<b>Limited Warranty</b>	<b>11</b>
<i>Pumping Made Simple</i>	5		
<i>Removing Ameda HygieniKit from Breast</i>	6		
<i>Troubleshooting</i>	6		

## SAVE THESE INSTRUCTIONS

### Before Every Use

Always wash hands well with soap and water before handling the Ameda HygieniKit Milk Collection System or other Ameda products. Do not use solvents or abrasives.

Before using, check the parts for cracks, chips, tears, color changes or breakdown. If seen, they can affect the function of the product. If parts need replacing due to defects, please call:

*In USA: 1.866.99.AMEDA(26332)*

*In Canada: 1.800.263.7400*

*In other countries: Call your local distributor or the location where you purchased your pump.*

*For a list of distributors in your country, visit [www.ameda.com](http://www.ameda.com).*

### For Your Health and Safety


The Ameda HygieniKit Milk Collection System is a personal care item. For hygienic reasons, this product should never be shared between mothers or resold from one mother to another.

# Instructions for the Ameda® HygieniKit® Milk Collection System

## Intended Use

The HygieniKit Milk Collection System when used with an Ameda electric breast pump is intended to express and collect milk from the mother's breast, to alleviate engorgement of the breast, maintain the ability of lactation, and provide mother's milk for future feedings when separation of mother and baby occurs.

## Set Up and Assembly

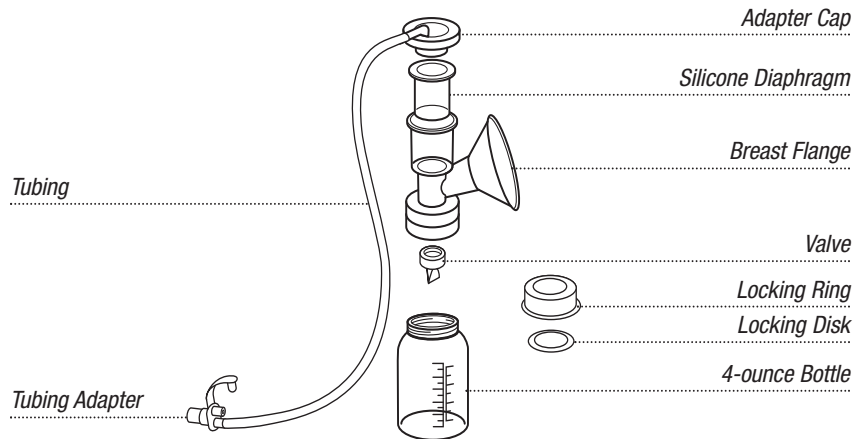## Cleaning

Unless told otherwise by a healthcare provider, follow these cleaning instructions: Always wash hands well with soap and water before handling the Ameda HygieniKit and other Ameda products. *Do not use solvents or abrasives.*

## Before First Use




If your Ameda HygieniKit is marked "STERILE," open and use.

If your Ameda HygieniKit is **NOT** marked "STERILE," before first use:

- Boil all parts—except tubing, adapter cap, and tubing adapter—for 20 minutes.
- Remove all parts from water right after boiling.
- Allow to air dry before assembly and use.

## After Every Use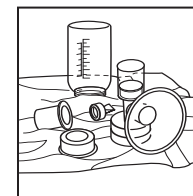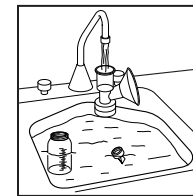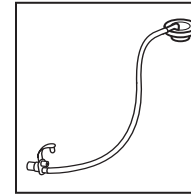

- Take apart the Ameda HygieniKit to clean.
- Rinse any milk residue from pump parts with cool water.
- Wash all parts—except tubing, adapter cap, and tubing adapter—in warm, soapy water.
- Rinse well with warm, clean water.
- Allow parts to air dry on clean towel or drying rack before assembly and next use.
- Cover parts with clean towel if not planning to use again right away.

Breast flanges and bottles can be washed on the upper rack of a standard household dishwasher away from heating elements. Wash diaphragms and white valves gently by hand. **Do not insert anything into valve while cleaning.**


## Sterilizing in the Hospital

If your hospital's protocol requires sterilization, autoclave at 273°F (133°C) for 5 minutes. Sterilize only pump parts that touch the milk, not the adapter cap, tubing adapter and tubing. Product should not be shared with other patients. It is intended for single patient use (repeated use by same patient). After sterilization and before using, check the parts for cracks, chips, tears, color changes or breakdown.

## Using the Ameda HygieniKit Milk Collection System


**ALWAYS WASH HANDS WELL WITH SOAP AND WATER BEFORE PUMPING.**

### Attaching to Any Ameda Electric Breast Pump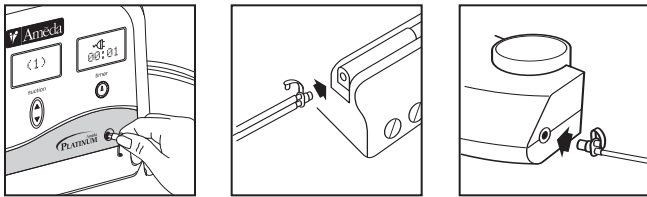


The HygieniKit may be used to single or dual pump with any Ameda electric breast pump. Simply insert the tubing adapter into the tubing adapter port.

### Single and Dual Pumping

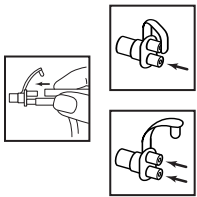**To single pump:**  
Remove one tube and close tubing adapter.

**To dual pump:**  
Both tubes should be attached to tubing adapter.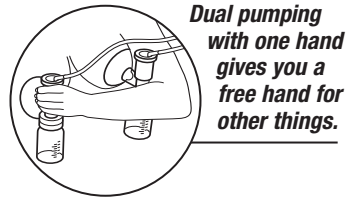

**Dual pumping with one hand gives you a free hand for other things.**

### Putting Ameda HygieniKit to the Breast


To begin pumping:

- Center the nipple in the breast flange.
- Fill the breast flange with the breast to create an air seal.
- Angle breast flange down a bit so milk flows into bottle.

### Checking Flange Fit


Some Ameda HygieniKits include one size breast flange, with an opening of 25mm. Other Ameda kits include three flanges sizes (25, 28.5, and 30.5mm). Many mothers do well with the 25mm standard size breast flange, but you may need a larger flange (30.5mm) or insert (28.5mm) if pumping is uncomfortable or unproductive, even on low suction. If the standard flange is too tight, a larger flange or insert may feel better and pump more milk. Flange fit can change as the breast changes after birth and with pumping. Use the size that feels best and gives the best results. If the 30.5mm size is not large enough, contact your Ameda retailer for the Ameda Custom Breast Flanges 32.5/36mm (Part No. 17358).

To check your flange fit, watch your nipple during pumping.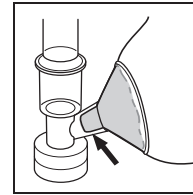

#### Good Fit

If you see space around your nipple, you have a good fit.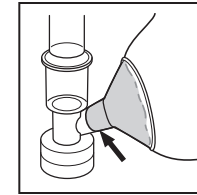

#### Too Tight

If your nipple rubs against the sides of the flange, it is too tight.

If your kit includes larger flanges, to use them:

- Replace standard flange with larger flange
- Move valve to larger flange
- See instructions to assemble other parts
- Try the larger flange with and without the insert on each breast until you find the size that feels better and/or pumps more milk

If your kit does not include larger flanges, contact your Ameda retailer for Custom Breast Flanges 28.5/30.5mm (Ameda Stock No. 17148P) or Custom Breast Flanges 32.5/36mm (Ameda Stock No. 17358).

Visit [www.ameda.com](http://www.ameda.com) for more information.

### Pumping Made Simple

Does a pump need to be set at the strongest suction to get more milk? No. Set your pump at the highest suction that feels good...and no higher. (If you're gritting your teeth, it's up too high!)

### Milk Release or 'Let-Down':

Pumping milk is not like sucking liquid through a straw. With a straw, the stronger you suck, the more liquid you get. When pumping, the milk comes only when a milk release (otherwise known as a "let-down") happens. Without a milk release, most milk stays in the breast.

What is a milk release?

- Hormones cause muscles in the breast to squeeze and milk ducts to widen, pushing the milk out.
- Some mothers feel tingling. Others feel nothing.
- A milk release can happen with a touch at the breast, hearing a baby cry, or even by thinking about your baby. Feeling your baby's soft, warm skin against yours can cause a milk release.
- Feelings of stress, anger, or upset can block milk release.

### Using Your Senses:

While breastfeeding, most mothers have three or four milk releases without knowing it. To get more milk with your pump, you need more milk releases. But, you may need some help at first until your body learns to respond to your pump like a baby.

To trigger more milk releases, use your senses. One or two senses may work better than the others, so try them all to find out which work best for you.

- **Mind/Feelings:** Close your eyes, relax, and imagine your baby breastfeeding. Think about how much you love your baby.
- **Sight:** Look at your baby or your baby's photo.
- **Hearing:** Listen to a tape of your baby cooing or crying. If you're away, call and check on your baby. Or, call someone you love to relax and distract you.
- **Smell:** Smell your baby's blanket or clothing.
- **Touch:** Apply a warm cloth or gently massage your breasts.
- **Taste:** Sip your favorite warm, non-alcoholic drink to relax you.

### Using Pump Controls:

Check to see if your pump has both SUCTION/VACUUM and CYCLE controls. If so, use your senses as you adjust the controls during pumping to get more milk faster. Watch your milk flow as you pump. Use it as your guide.

1. Set SUCTION/VACUUM to the highest setting that feels good.
2. Set CYCLE on the fastest setting.
3. When milk flows, turn CYCLE down close to the slowest setting.
4. When the milk flow slows to a trickle, return to fast CYCLE and use your senses.
5. Repeat, using fast CYCLE to trigger milk releases and slow cycles to drain.

### Pumping Length:

Talk to your healthcare provider about how long you should pump. In most cases, 10 to 15 minutes per breast is long enough.

- If single pumping, 20 to 30 minutes total.
- If dual pumping, 10 to 15 minutes total.

### Removing Ameda Milk HygieniKit from Breast

After pumping:

- Turn off pump.
- Insert a finger into breast flange(s) to break suction.
- Remove Ameda kit from breast(s).

### Troubleshooting

If milk drips too slowly into bottle:

- If pump has a CYCLE control, set pump to faster CYCLE setting.
- Replace silicone diaphragm(s) and valve(s).

If pumping causes discomfort:

- Set pump to lower SUCTION/VACUUM setting.
- Check pump fit. Larger flanges may be needed.

If suction is low:

- Check that valve(s) is/are pushed onto breast flange(s).
- Check valve(s) for tears or stretches. Replace if needed.
- If single pumping, check tubing adapter to be sure side not in use is closed.
- Check tubing adapter for cracks. If seen, replace.
- Set pump to higher SUCTION/VACUUM setting.

The valves are vital for proper suction. To work well, they must be intact and look closed when pump is not in use.

### To prolong valve life:


- When removing from breast flange, pull gently from thick sides, not thin edge.
- Do not stick anything into valves during cleaning. Just swish them around in warm, soapy water, rinse well in warm water, and air dry.

## Converting to an Ameda One-Hand Manual Breast Pump

**ALWAYS WASH HANDS WELL WITH SOAP AND WATER BEFORE PUMPING.**


### Assembly

Use this section if your One-Hand handle assembly (Part no. 17145) is part of an Ameda HygieniKit or has been purchased separately, or has been converted with the Manual to Electric Conversion kit (Part no. 17141) for use with an Ameda electric pump.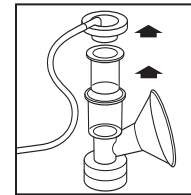


1. Unsnap and remove adapter cap.
2. Remove diaphragm from upper chamber of breast flange.


Remove plastic covering on handle assembly.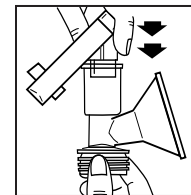

3. Push piston into upper chamber of flange.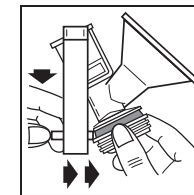

4. Pull handle down and snap handle collar around bottom of flange.  
(See shaded area in diagram above.)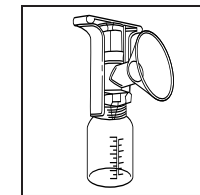

5. Screw flange onto bottle.  
*Handle is on correctly if top of handle is a straight line.*

## Using the One-Hand Manual Breast Pump

### Pumping Made Simple

Use both fast and slow squeezes. Watch your milk flow as you pump. Use it as your guide.

1. Use a few minutes of fast, light squeezes to trigger milk flow.
2. When milk flows, use slower, stronger squeezes. (Never so hard that it hurts.)
3. For easier pumping, while milk is flowing “squeeze and hold.”
4. When milk flow slows, use faster squeezes again.
5. When milk flows again, use slower, stronger squeezes.
6. Repeat fast then slow squeezes to pump more milk faster.
7. Change breasts every 5-7 minutes for a total of 20 to 30 minutes.

Pumping with fast and slow squeezes is like a baby at the breast. Baby starts fast, then slow, and repeats. This two-speed action may help you pump more milk faster.

### Removing Ameda One-Hand Breast Pump from Breast

After pumping:

- Insert a finger into breast flange to break suction.
- Remove One-Hand Breast Pump from breast.

### Troubleshooting

If milk drips into bottle too slowly:

- And you’re using “squeeze and hold,” release the handle.
- If not using “squeeze and hold”, release handle more often.

If pumping causes discomfort:

- Use gentler handle squeezes.
- Check flange fit. A larger flange may be needed.

If suction is low:

- Check that valve is on fully.
- Check valve for tears or stretches. Replace if needed.
- Remove piston seal, stretch out on flat surface, and replace on piston.

The valves are vital for proper suction. To work well, they must be intact and look closed when pump is not in use.

### To prolong valve life:

- When removing from breast flange, pull gently from thick sides, not thin edge.
- Do not stick anything into valves during cleaning. Just swish them around in warm, soapy water, rinse well in warm water, and air dry.

## Milk Storage and Handling

### Containers

– ALWAYS DATE CONTAINERS –

Human milk should be stored in a dated, clean, sealed container. Glass or hard plastic containers can be used.

### Guidelines for Storing and Thawing Human Milk

Storage Time for Human Milk*	Deep Freeze	Refrigerator Freezer (variable 0°F/-18°C)	Refrigerator (39°F /4°C)	Cooler with Frozen Ice Packs (59°F/15°C)	Room Temperature	
	(0°F/-18°C)				(66°F–72°F) (19°C–22°C)	(72°F–79°F) (22°C–26°C)
<b>Fresh</b>	12 Months	3–4 Months	8 Days	24 Hours	6–10 Hours	4 Hours
<b>Frozen, Thawed in Fridge</b>	Do Not Refreeze	Do Not Refreeze	24 Hours	Do Not Store	4 Hours	4 Hours
<b>Thawed, Warmed, Not Fed</b>	Do Not Refreeze	Do Not Refreeze	4 Hours	Do Not Store	Until Feeding Ends	Until Feeding Ends
<b>Warmed, Fed</b>	Discard	Discard	Discard	Discard	Until Feeding Ends	Until Feeding Ends

\* Storage times may vary for premature or sick babies. Sources: Jones, F. and M.R., Tully, *Best Practice for Expressing, Storing and Handling Human Milk in Hospitals, Homes and Child Care Settings*. Raleigh, NC: Human Milk Banking Association of North America, 2006; Mohrbacher, N. *Breastfeeding Answers Made Simple*. Amarillo, TX: Hale Publishing, 2010.

### Thawing/Warming Milk

Thaw frozen milk in refrigerator. Or, thaw or warm milk by placing container in cool then warm water.

– Do Not Microwave Human Milk –


Microwaving changes the milk. It also creates hot spots in the milk that can burn the baby.

## Flexishield Areola Stimulator


The Flexishield Areola Stimulators (Ameda Stock No. 17212):

- Is a flexible silicone breast flange insert.
- Gives a soft feel against the skin while pumping.
- Compresses the breast while pumping to help trigger milk release.
- Is optional with the Ameda One-Hand Breast Pump and the Ameda HygieniKits.
- Can be bought separately.
- Narrows the breast flange, so works best with mothers with small nipples.
- Fits only the standard 25mm breast flange (see “Checking Flange Fit”).

To use the Flexishield areola stimulator:

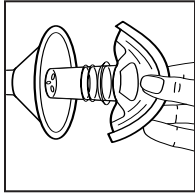
1. Insert Flexishield stimulator into breast flange. Flexishield stimulator's rings should touch inside walls of flange.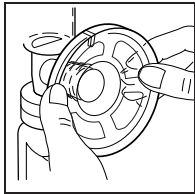


2. Attach Flexishield stimulator to flange by pushing edge over rim of flange. Use palm or thumb and forefinger.


## Replacement Parts

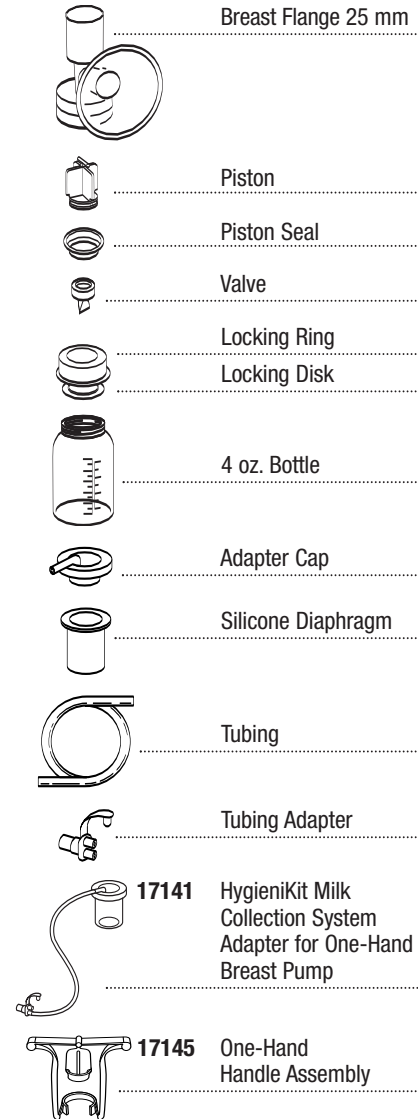17152 Dual HygieniKit Milk Collection System with One-Hand Breast Pump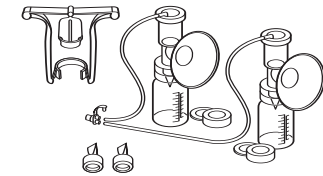

17449 Dual HygieniKit with CustomFit Flange System and One-Hand Breast Pump Adapter

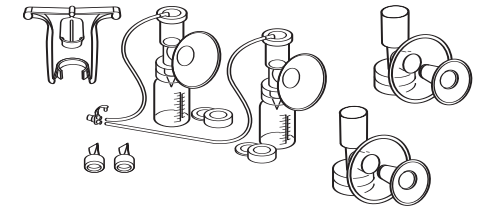### Products not shown

- 17148P Custom Breast Flanges (28.5/30.5 mm)
- 17150 Single HygieniKit Milk Collection System with One-Hand Manual Breast Pump Adapter
- 17151 Single HygieniKit Milk Collection System
- 17153 Dual HygieniKit Milk Collection System with One-Hand Manual Breast Pump Adapter & Flexishield Areola Stimulator
- 17155 Dual HygieniKit Milk Collection System
- 17212 Flexishield Areola Stimulator (1 Pr)
- 17242 Store 'N Pour™ Breast Milk Storage Bags (20 Bags + 2 Adapters)
- 17243 Store 'N Pour Breast Milk Storage Bag Refills (40 Bags)
- 17358 Custom Breast Flanges (32.5/36 mm)
- 17448 Single HygieniKit CustomFit Flange System with One-Hand Breast Pump Adapter

## Limited Warranty:

The Ameda HygieniKit Milk Collection System (“Product”) is warranted to the original user (“Customer”) only. Use of the Product by more than one person will void this express limited warranty.

Evenflo Company, Inc. (“Evenflo”) warrants that it will convey to the Customer good title to the Product. For ninety (90) days from the date of original purchase of this Product, Evenflo warrants the Product to the Customer against defects in material or workmanship. Evenflo’s sole obligation under this express limited warranty shall be to repair or replace, at Evenflo’s option, any Product that is determined to be defective by Evenflo and determined to be covered by this limited warranty. Repair or replacement under this limited warranty is the sole and exclusive remedy of the Customer. Proof of purchase in the form of a receipted invoice or bill of sale evidencing that the product is within the warranty period must be presented to obtain warranty service. This express limited warranty is extended by Evenflo ONLY to the Customer and is not assignable or transferable to subsequent purchasers or end users of the Product. For warranty service contact Ameda at 1.866.99.AMEDA (26332).

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE FOR THE KIT IS HEREBY DISCLAIMED. EVENFLO SHALL NOT BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL OR PUNITIVE DAMAGES OF ANY KIND FOR BREACH OF THE EXPRESS LIMITED WARRANTY ON THE KIT OR ANY WARRANTY IMPLIED BY OPERATION OF LAW. OTHER THAN THE EXPRESS LIMITED WARRANTY SET FORTH ABOVE, THERE ARE NO OTHER WARRANTIES THAT ACCOMPANY THIS KIT AND ANY ORAL, WRITTEN OR ANY OTHER EXPRESS REPRESENTATION OF ANY KIND IS HEREBY DISCLAIMED.

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MOM INSPIRED. HOSPITAL TRUSTED.

Distributed in USA by:  
**Evenflo Company, Inc.**  
 1801 Commerce Drive  
 Piqua, Ohio 45356  
 1.866.99.AMEDA (26332)

Distributed in Canada by:  
 Distribué au Canada par :  
**Mothers Choice Products, Ltd.**  
 2133-11871 Horseshoe Way  
 Richmond, BC V74 5h5  
 1.800.604.6225

Distributed in Australia by:  
**Advantage Health Care**  
 34 Percy Street  
 Mordialloc, VIC 3195  
 61 (3) 9580 9288

[www.ameda.com](http://www.ameda.com)

**Distribution in other countries:**  
 For product information or feedback, visit  
[www.ameda.com/ameda-solutions/international-distributors](http://www.ameda.com/ameda-solutions/international-distributors)

**Dual HygieniKit® Milk Collection System**

with One-Hand Breast Pump Adapter  
 and Flexishield™ Areola Stimulators

**One-Hand  
 Manual  
 Breast Pump**

CE 0086

EMERGO EUROPE  
 Molenstraat 15  
 2513 BH, The Hague  
 The Netherlands  
 Phone: +31.70.345.8570  
 Fax: +31.70.346.7299

Evenflo Company, Inc.  
 1801 Commerce Drive  
 Piqua, Ohio 45356  
 USA

Ameda, HygieniKit, CustomFit Flange System, Flexishield, Store 'N Pour  
 and associated logos are trademarks of Evenflo Company, Inc.

©2011 Evenflo Company, Inc.

25700716-1211

**HygieniKit Milk Collection Systems and  
 One-Hand Manual Breast Pump Instructions**

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